



PERSONAL inLOOP

inLOOP products are made exclusively in the USA. Some electronic parts are not produced in the US and have to be imported.

SPECIFICATIONS

12 watt Personal amplifier includes the revolutionary inLOOP noise canceling sound chamber with a built in microphone. 1 analog audio line input.

DIMENSIONS

Amplifier 3" x 3" x 5"
Package 4" x 4" x 16"
2 lbs.

CONTENTS

inLOOP PIP amplifier
12 volt 1 amp adapter
inLOOP Pad
3.5 mm analog cable
3.5 mm to RCA adapter


WARRANTY

The Personal inLOOP amplifier is warranted for 5 years; excluding damage or failure caused by misuse, neglect or excessive electric current or voltage fluctuations.

569 South State Street
Shelby, MI 49455
info@inLOOP.tv
(231) 798-2399



inLOOP

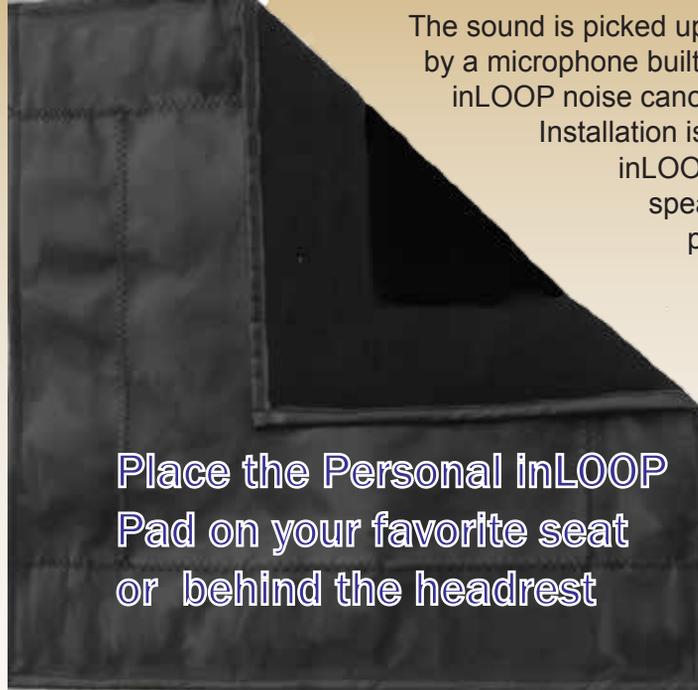

Place the Personal inLOOP Pad on your favorite seat or behind the headrest

The sound is picked up from the TV speaker by a microphone built into the revolutionary inLOOP noise canceling sound chamber. Installation is as simple as setting inLOOP in front of the audio speaker, plugging it into a power outlet, adjusting the volume and sitting on the Personal inLOOP Pad

ADJUST the volume by turning the knob on top of inLOOP. Clockwise makes it louder counter-clockwise makes it quieter.

Connect the sound source

CABLE CONNECTION

If the sound source has a red and white (or black) output, install the audio cable from there to the inLOOP "INPUT".



OR MICROPHONE

The sound can be picked up from the TV speaker by the microphone built into the revolutionary inLOOP noise canceling sound chamber. Installation is as simple as placing inLOOP in front of the TV speaker.



FOR DIGITAL APPLICATIONS UPGRADE TO inLOOP 600

WARNING This device creates heat as part of the loop process. To insure proper cooling make sure inLOOP is not covered and standing vertical like the photos above.